

INSTRUCTIONS FOR USE

MAIS HIGH PRESSURE EXTENSION LINE™

Intended Use:

Tube Used to deliver fluid in body with High Pressure.

Materials Used:

PVC, Polyethylene, Polycarbonate.

Indications:

- It is connected to any system for delivery of fluids or drugs simultaneously or intermittently.
- This Device can be used by itself or as an accessory of administration / Extensions sets.

Contra Indication:-

- Administration of highly viscous fluids.
- Not to be used in patients with known hypersensitivity to any of the materials used.

Instructions for use:

- Peel open the pouch.
- Unscrew the protective cap of female luer.
- Connect male portion of I.V. Set/bag with female luer.
- Remove protective cap of male luer of tubing.
- Hold male luer of tubing upright and open clamp of I.V. Set / bag and allow the solution to displace air entirely from tubing.
- Close Clamp.
- Connect male luer of tubing to venipuncture device.
- Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or infections.

Warnings:

- Store in a cool & dry place. Protect from excessive heat or direct sunlight.
- Read instructions before use. The product should be used according to the instructions for use.
- This product should be used only by a qualified Doctor or a Paramedic.
- Check the product and package integrity and also the expiry date prior to use.
- The product is guaranteed sterile, non-pyrogenic and non-toxic if package is not opened or damaged.
- Use immediately after opening the individual packing.
- Use ONLY standard Luer connection devices; non-standard connectors can damage the product.
- For single use only. Discard after use. Do not recycle, clean or resterile. MAIS INDIA disclaims any responsibility for possible consequences resulting from improper use.

Caution:

- Do not use if package is damaged.
- Discard after single use.

Target Age Group: For All age groups

Duration of Contact of Device : Up to 72 Hours.

Storage condition: Store at a temperature between +10°C and +40°C, in a dry place and away from sunlight.

Device Life: 05 Years


	CE Notified Body No.		Keep Dry		Non-Pyrogenic
	Catalogue Number		Caution		Latex Free
	Lot No./ Batch No.		Do Not Use If Package is Damaged		Do Not Reuse
	Manufacturer		Sterilized By Ethylene Oxide		Do Not Resterilize
	Date of Manufacturing		Authorised Representative In The European Union		Medical Device
	Use By / Expiry Date		Single Sterile Barrier System		Unique Device Identifier
	Consult Instructions for Use		Keep Away From Sunlight		Country of manufacture
	Temperature limit (Upper 40°C and lower 10°C)				

Manufactured by:
Mais India Medical Devices Pvt. Ltd.
 525-P, Sector-37, Pace City-II,
 Gurgaon, Haryana-122001 (INDIA)
 Mfg. Lic. No.: MFG/MD/2019/000037


Obelis S.A.
 Boulevard Général Wahis 53, 1030 Brussels, Belgium

 0123

Edition: 02, Rev.: 00_02.02.2026

REV BY	Q.A			TITLE : Instructions for Use of Mais High Pressure Extension Line (P.M. Line)		SHEET No. 1/1	APPROVED BY:
	Q.C					DRW. No.: I07001-IFU-000	
	Pro.						
CHD BY	Mkt.			COLOR OF PRINT: Blue - Pantone 293 C		EDITION: 02, REV.: 00	
DGN BY	Gopal Singh		DATE: 02.02.2026			Size : 148 x 210 mm	
SCALE-N.T.S TOLERANCE : ±2 mm				REVISION DETAILS:			
MAIS INDIA MEDICAL DEVICES PVT. LTD. 525-P SECTOR-37, PACE CITY II, GURGAON, HARYANA-122001 (INDIA)				Edi. 02, Rev. 00: The humidity limitation symbol has been removed and EC-REP has been changed to EU-REP and Symbol Updated.			