



INSTRUCTIONS FOR USE

MAIS-NEB™ Nebulizer Mask

Device Description:

The Nebulizer Mask is a non-invasive respiratory device used to deliver aerosolized medication to patients via inhalation. The mask covers the nose and mouth and is connected to a nebulizer chamber that converts liquid medication into a fine mist using compressed air or oxygen.

Intended Use:

The Nebulizer Mask is intended for administration of aerosolized medications to patients requiring inhalation therapy for respiratory conditions.

Intended Users:

- Healthcare professionals
- Trained caregivers
- Patients under supervision (home-care use)

Intended Patient Population:

- Adult/ Pediatric/ Neonatal (size specific)

Sterility: Sterile (Sterilized by EO).

Indications for Use:

Used for delivery additional medication in conditions such as:

- Asthma
- Chronic Obstructive Pulmonary Disease (COPD).
- Bronchitis
- Other respiratory disorders as prescribed by a physician.

Contraindications:

- Patients with known hypersensitivity to materials used in the device.
- Patients unable to tolerate a face mask.
- Do not use for patients requiring invasive ventilation.

Principle of Operation:

Compressed air or oxygen passes through the nebulizer chamber, converting liquid medication into aerosol particles, which are inhaled by the patient through normal breathing.

Components:

- Face Mask
- Nebulizer Chamber
- Connector
- Elastic Head Strap
- Medication Cup
- Oxygen / Air Supply Tubing (if supplied)

Instructions for Use:

1. Wash hands thoroughly before handling the device.
2. Remove the Nebulizer Mask from the packaging and visually inspect for damage.
3. Fill the nebulizer chamber with prescribed medication as directed by a healthcare professional.
4. Assemble the nebulizer chamber and connect it to the mask.
5. Connect the tubing to the compressed air or oxygen source.
6. Place the mask securely over the patient's nose and mouth.
7. Adjust the elastic strap to ensure a comfortable fit.
8. Turn on the gas supply/nebulizer and allow the patient to breathe normally.
9. Continue therapy until the medication is completely nebulized or as prescribed.
10. Switch off the gas supply and remove the mask.

Warnings:

- For single patient use only.
- Do not use if the device or packaging is damaged.
- Use only as prescribed by a physician.
- Do not modify the device.
- Keep away from fire and ignition sources when used with oxygen.

Precautions:

- Ensure correct mask size for the patient.
- Monitor the patient during therapy.
- Do not overfill the medication cup.
- Replace the device if it shows signs of wear or damage.

Cleaning and Disposal:

- Device is single-use unless otherwise stated
- Dispose of the device after use according to hospital or local biomedical waste guidelines
- Do not attempt to re-sterilize

Storage Conditions:

- Store in a clean, Cool & dry place. • Protect from direct sunlight.

Shelf Life: 5 Years.

Symbols Used on Label:

	Consult Instruction For Use		Do Not Reuse		Unique Device Identifier
	Catalogue Number		Do Not Resterilize		Sterilised By Ethylene Oxide
	Lot No./Batch No.		Caution		Single Sterile Barrier System
	Manufacturer		Do Not Use If Package Is Damage		Medical Device
	Date Of Manufacturing		Keep Dry		Non-Pyrogenic
	Use By / Expiry Date		Keep Away from Sunlight		

Manufactured by:
Mais India Medical Devices Pvt. Ltd.
 525-P, Sector-37, Pace City-II,
 Gurgaon, Haryana-122001 (INDIA)
 Mfg. Lic. No.: MFG/MD/2019/000037

ISO & CE Certified Company

REV.: 01_02.01.2026

REV BY	Q.A		TITLE : Instructions for Use of MAIS-NEB (Nebulizer Mask)		SHEET No. 1/1	APPROVED BY:
	Q.C					
	Pro.					
CHD BY	Mkt.		For Party Code 000		DRW. No.: A03001-IFU-000	
DGN BY	Gopal Singh	DATE: 02.01.2026	COLOR OF PRINT: Blue 293c		REV.: 01	
SCALE-1:1	TOLERANCE : ±2 mm		REVISION DETAILS:		SIZE: 148 x 210 mm	
MAIS INDIA MEDICAL DEVICES PVT. LTD. 525-P SECTOR-37, PACE CITY-II, GURGAON, HARYANA-122001 (INDIA)			Rev.: 01 The pin code in the address has been changed from 'Gurgaon-122001, Haryana' to 'Gurgaon, Haryana-122001', and the symbols have been updated.			